Searching for a Template Group 2.0

When you search the AIM Template Service for a template group that you want to modify in ATB, you import it at the same time. You do not have to take a separate step to download the template group. Once you find the template group in the search results, click it to begin using it in ATB.

1. Click the Template Groups tab.

2.	Below the left panel, click		Onknown At	Unknown Attachment		and then select Search Groups.						
	The Template Groups Search page appears.											
	O O Template Groups Search											
	Co	Container Name ntainer Description Container Autho	1:	Dise Body F Tag Na Tag Va	ime :							
	Name	Authors	Search Description	Reset Ca	Disease	Body Part	UID					
		,,) <u> </u>								

3. Narrow your search by entering information into the fields. If you do not enter any information into the fields, all template groups available in ATS appear, and it may take longer to see the results of your search. The following fields are available.

Field	Description				
Container Name	Template group name				
Container Description	Template group description				
Container Author	Person who created the template group in the AIM-enabled application				
Disease	Disease that is the topic of the templates in the template group				
Body Part	Body part that is the topic of the templates in the template group				
Tag Name	A name associated with this template group				
Tag Value	The value associated with the tag name for this template group				

4. Click Search.

All template groups matching the search criteria you entered appear.

		Temp	late Groups Search			
Container Name : Container Description : Container Author :			Disease : Body Part : Tag Name : Tag Value :]]]
News		Search	Reset Cancel			
Name	Authors	Description	Tags	Disease	Body Part	UID
	kirbyju	TCGA Breast Research Group te	Body Part	Challen and a second	Breast	1.3.6.1.4.1.1451
O ATB-VASARI-Exa	AIM Team	This template is used by an ap	Body Part, Disease	Glioblastoma	Brain	2.25.234308535
O TCGA Renal	kirbyju	Template for TCGA Renal Rese				2.25.215225874
O MR-Cardiac	Northwestern	MR Cardiac				2.25.822196569
		MR Knee	Source, Modality, Anatomic			2.25.208977181
🔾 TEST 3 – 1	AIM Team - 1	Test three – 1	Varsari, Varsari			2.25.274792958

5. Click the name link for a template group. ATB imports the template.