

Searching for a Template from AIM Template Service 2.0

You can search the AIM Template Service for a template that you can customize in ATB. Once you find the template you want to use in the search results, you can [import](#) it. ATB 2.0 automatically converts older ATB templates to the latest schema version upon import.

1. Click the **Templates** tab.
2. Below the left panel, click

Unknown Attachment

 and then select **Search Templates**.
The Template Search page appears.

Template Search

Template Name :

Template Code Desc :

Template Description :

Template Author :

Modality :

Disease :

Body Part :

Tag Name :

Tag Value :

Search

Reset

Cancel

Name	Authors	Description	Tags	Disease	Body Part	Modality	UID
------	---------	-------------	------	---------	-----------	----------	-----

3. Narrow your search by entering information into the fields. If you do not enter any information into the fields, all template groups available in the AIM Template Service appear, and it may take longer to see the results of your search. The following fields are available.

Field	Description
Template Name	Template name
Template Code Desc	Template code description
Template Description	Template description
Template Author	Person who created the template in the AIM-enabled application
Modality	Any of the various types of equipment or probes used to acquire images of the body, such as radiography, ultrasound and magnetic resonance imaging
Disease	Disease that is the topic of the templates in the template group
Body Part	Body part that is the topic of the templates in the template group
Tag Name	A name associated with this template
Tag Value	The value associated with the tag name for this template

4. Click **Search**.
All templates matching the search criteria you entered appear.

Unknown Attachment